



LIMITED LIFETIME WARRANTY

Revolution Mobility is focused on providing quality products. This product is designed and built to precise standards and is carefully inspected prior to shipment. This product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This warranty does not cover product failures due to owner misuse or negligence or normal wear and tear. This warranty does not extend to non-durable components such as rubber accessories which are subject to normal wear and tear and does require periodic replacement. In the event of a defect, Revolution Mobility will at our option replace or repair the product.

If you have any questions please contact an authorized Revolution Mobility Dealer.



3072 Inland Empire Blvd. • Ontario, CA 91764
866.980.2259 • 909.980.2269 fax
www.RevolutionMobility.com

MANUAL WHEELCHAIRS



INSTRUCTIONS AND WARRANTY



MANUAL WHEELCHAIRS

PLEASE READ THIS MANUAL CAREFULLY TO ENSURE SAFE USAGE AND MAINTENANCE

HOW TO OPERATE AND MAINTAIN

Wheel locks: Apply both wheel locks before getting in or out of chair, leaning forward in chair, while in an elevator or in a wheelchair lift. Wheel lock should be adjusted to at least 1/8 inch when locked. Push handle completely forward to lock. Adjustment to locks are made by tightening the nut on the carriage bolt. Embed lock shoe at least 1/8 inch by sliding clamp toward the rear of the wheel while handle is engaged in the locked position. Test for locking before actual use.

Tires: Solid rubber tires need minimal maintenance but should be cleaned with a damp cloth occasionally. If tires become severely worn out or if cracking appears please replace tires.

Wheels: Large rear wheel should be inspected at least once a month to ensure proper tightness. If wheel wobbles, release lock nut and tighten. Test tightness by leaning chair to one side.

Front casters: Casters should be checked periodically and lubricated occasionally. Lubricate by removing fork and greasing caster stem bearings. Lubricate wheel bearings by removing wheel from fork. Reassemble fork and wheel tightening till no play remains but wheels roll freely. Swing away footrest: To remove, release lock by pushing on the front rigging lever. Swing away foot rest/leg rest out and sideways. Lift to remove. To adjust foot rest or leg rests, user should be sitting in wheelchair with feet on foot plates. Loosen bolt on the clamp around support assembly. Raise the lower footplate to the desired position that provide maximum support for thighs. Front rigging should not be so low as to put uncomfortable pressure on the thighs

at front of seat upholstery. Footrest should not be lower than 2-1/2 inch from the ground to ensure clearance.

Elevating leg rest: Lift foot plate on the leg rest assembly to desired position. Release leg rest assembly and it will automatically lock in place. To lower push out leg rest lever and lower foot plate to desired position.

Metal components: Wipe all metal parts with a clean soft cloth. If exposed to water dry immediately. Do not use abrasive cleaners.



Removable arms: Remove arms by depressing spring button and lifting arms out of front and rear sockets.

To open chair: Open wheelchair by tilting to one side and pushing down on seat till it opens fully

To close chair: First fold foot plates to vertical position. Grasp seat with both hands at front and back edge and lift. Or tilt on one side and lift up seat rail.

Precautions and warnings

1. Always transfer wheelchair to a vehicle manufacturers approved seat and use safety restraint. This wheelchair has not been approved as a seat for vehicles of any kind.
2. Do not operate wheelchair on roadways, streets or anywhere with traffic.
3. Do not attempt to operate on inclines greater than 10% without assistance.
4. Always apply brakes when getting in or out of chair. Do not apply weight to foot rests when getting in and out of chair.
5. Do not lower footrest to lower than 2-1/2 inch above floor.
6. Before reaching or leaning forward, rotate caster wheels to front of chair to engage wheel locks to prevent wheelchair from tipping.
7. Do not apply weight to the wheelchair back since this can cause tipping. Do not over extend arms behind back—reach back as far as you can without changing position in seat.
8. DO NOT stand on wheelchair.
9. DO NOT use wheel locks to slow your descent down an incline.
10. DO NOT abruptly change direction when going down an incline as this can cause tipping